Multi-Purpose Opener

VM915

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Multi-Purpose Opener from Aidapt. The Multi-Purpose Opener is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Multi-Purpose Opener is a useful kitchen aid for those with limited dexterity. It has an adjustable steel band which clamps around a jar or bottle top and also incorporates a bottle top opener and ring can pull.

INSTRUCTIONS FOR USE

The Multi-Purpose Opener is simple to use:

- **To Open Jars:** Grip the Multi-Purpose Opener with the ergonomically-designed handle and place the steel band over the jar lid, tighten the band using the turning knob. Position the Multi-Purpose Opener so that it grips the jar lid and pull/push the handle to twist the jar lid off the jar.
- **To Remove Bottle Caps:** Grip the Multi-Purpose Opener with the ergonomically-designed handle and place the product over the jar lid. Position the Multi-Purpose Opener so that the central hole and metal lip are above/below the edge of the bottle tops. Pull the handle of the Multi-Purpose Opener up to flip the cap off the bottle.
- **To Open Can Lids:** Grip the Multi-Purpose Opener with the ergonomically-designed handle and position the hook below the edge of the can ring pull. Push the handle of the Multi-Purpose Opener down to pull the lid off the can.

CLEANING

Clean your Multi-Purpose Opener using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Multi-Purpose Opener can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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**Altai Germany (Altai Europe Ltd.)