Foot & Leg Cover for Scooter/Wheelchair Users

VA129C/VA129SS

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Foot & Leg Cover for Scooter/Wheelchair Users from Aidapt. The Foot & Leg Cover is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

NB. This equipment must be installed by a competent person.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the item.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

FITTING INSTRUCTIONS

The Foot & Leg Cover can easily be slipped on and off:

- Unzip the opening fully
- Put the Foot & Leg Cover on the seat of the scooter or wheelchair, with the opened sections spread to either side.
- Carefully position yourself (or the user) in the seat on top of the opened Foot & Leg Cover, with your feet placed firmly in the foot area
- Use the full length zip to close the Foot & Leg Cover up to the waist
- Hook elastic loops over pushign handles, where applicable.

CLEANING

Clean your Foot & Leg Cover using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

The Foot & Leg Cover can be cleaned in a washing machine at a low temperature, ideally on a wool or delicates setting.

After cleaning, hang the Cover somewhere warm and dry to air. Do not leave in direct contact with a heat source and do not put in a tumble dryer.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Holland (Altai Europe Ltd)