Adult Drinking Cup

VM922

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Adult Drinking Cup from Aidapt. The Adult Drinking Cup is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

This Drinking Cup has a long mouthpiece making it easy to hold and comfortable to use. This specially-designed cup is supplied with two tops, one with a small hole for drinking fluids and the other featuring a larger hole for more solid food.

INSTRUCTIONS FOR USE

The Drinking Cup is simple to use:

- For drinking fluids, pour the drink into the cup then attach the supplied lid with the smaller hole.
- For more solid food or meal replacement drinks, pour the fluid into the cup then attach the supplied lid with the larger hole.

For your safety:

- Do not overfill the cup.
- Make sure that the lid is snugly attached before use.
- The Drinking Cup is not recommended for hot drinks.

CLEANING

Clean your Drinking Cup using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Drinking Cup can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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Altai Germany (Altai Europe Ltd.)