



Weekday Pill Dispenser



VM929A

Usage and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Weekday Pill Dispenser from Aidapt.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.
- **As with all pills or medicines, keep out of reach of children.**

INTENDED USE

The Weekday Pill Dispenser is for use as a general pill sorter or pill dispenser during holidays. It has seven clearly labelled individual removable compartments for each day of the week, with each day having four compartments: morning, noon, evening, and night.

Instructions for Use

1. To open the daily compartment, locate the day you wish to open and press the button.
2. To close, push the compartment firmly in until you feel or hear a slight click as it clicks into place.
3. Ensure you do not overfill the compartments as this may cause damage to the pills, or prevent the lid from closing properly.

CLEANING

Clean your Weekday Pill Dispenser using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure that you wipe the equipment dry after cleaning.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com
Email: • sales@aidapt.co.uk