



Plastic Bath Step



VR278D

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Plastic Bath Step. The Plastic Bath Step is manufactured from the finest materials and components available, when used correctly it is designed to give many years of reliable trouble-free service.

NB. Do not exceed the weight limit stated – doing so could put the user at risk.

NB. Weight limit: 120 kg (18.5 stone)

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.
- As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement, when using this equipment.

WARNINGS & USAGE

- Ensure that the step is dry and free from soap or oil film.
- Ensure that the back of the step is positioned against a solid upright surface to prevent the step from moving forward.
- Step centrally on the step with both feet.
- Test for stability.
- DO NOT step onto the edge of the step; this may cause it to tip. Always step onto the middle of the step top and ensure that your weight is evenly distributed.
- DO NOT jump onto or off the step.
- Use the step only when dry to reduce the risk of slipping and ensure that the surface beneath the step is dry before use.
- **NB. This is not a toy; ensure that children are supervised at all times.**

HEIGHT ADJUSTMENT & STACKING

Multiple Bath Steps may be used together to increase the height overall, by simply stacking two or three Bath Steps one on top of the other, ensuring that each step is fully pushed down onto the step below.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your Plastic Bath Step using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners or abrasive cleaning pads may seriously damage your product beyond repair.

USE OF BATH OILS

Caution should be observed if you are using any form of bath oils or oily skin preparation with your Bath Step as this may impair your natural contact skin resistance.

In the event of using any such oil, your Bath Step must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk