



## **Foam Bed Wedge**



**VG884**

## **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for deciding to purchase the Foam Bed Wedge from Aidapt. The Foam Bed Wedge is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

## **BEFORE USE**

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

## **INTENDED USE**

- The Foam Bed Wedge has been designed to offer improved comfort in an upright or reclined position.
- The Foam Bed Wedge can be used in two ways:
- To recline, position the large, flat surface on the bed with the deepest end of the wedge below your head and shoulders.
- To use the Foam Bed Wedge for support when seated upright, position the large, flat surface against a solid object such as a headboard or wall, with the widest end of the wedge on the bed.
- No fixings are required; the Foam Bed Wedge can simply be removed and stored when not in use.
- Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.

## **CLEANING**

Clean your Foam Bed Wedge using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the foam inner. The cover can be removed and washed at a low heat with similar colours.

After cleaning, leave the cushion and cover to dry in a warm, dry place, but not in direct contact with a heat source.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**

**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**

**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**