Extendable, Plastic-Handled Walking Sticks with Pattern

VP155JK/VP155JP/VP155JT

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Extendable Plastic Handled Walking Stick from Aidapt. These Walking Sticks are manufactured from the finest materials and components available, when used correctly they are designed to give many years of reliable trouble-free service.

The above Walking Sticks are height adjustable, with 5 height settings via a push button mechanism, a slip resistant rubber foot and convenient, integrated hanging strap.

IMPORTANT!

The user weight limit for this product is 100 kg (15 stone).

NB. DO NOT exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

HEIGHT ADJUSTMENT

1. Grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft.
2. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position.
3. Prior to use always ensure that the locking pin fully protrudes from the selected hole position and by applying downward pressure on the handle of the walking stick test that the walking stick has been assembled correctly.

CLEANING

Clean your Walking Stick using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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Altai Germany (Altai Europe Ltd.)