Scooter Covers

VA118SS (Large)/VA119SS (Medium)

Usage and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for deciding to purchase the Scooter Cover from Aidapt.

NB. This equipment must be installed by a competent person.

INTENDED USE

The Scooter Cover has been designed to entirely cover 3 or 4 wheeled mobility scooters when not in use and provide protection from the elements.

FITTING INSTRUCTIONS

The Scooter Cover can easily be slipped on and off as necessary; the opening is elasticated for a secure fit:

- Remove all packaging; take care not to use sharp tools such as scissors or knives to open the packaging as this may result in damage to the Scooter Cover
- ***NB. Please dispose of all packaging safely***
- Starting at either the front or back end of the scooter, drape the Scooter Cover over the entire scooter, ensuring that the elasticated opening of the cover is around and below the scooter as much as possible all around.
- ***NB. Before fitting the Scooter Cover you must ensure that the scooter is switched off and any secondary brake is applied, as necessary. The Scooter Cover must not be used while the Scooter is in use.***

CLEANING

Clean your Scooter Cover using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

The Scooter Cover can be cleaned in a washing machine at a low temperature, ideally on a wool or delicates setting.

After cleaning, hang the Cover somewhere warm and dry to air. Do not leave in direct contact with a heat source and do not put in a tumble dryer.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Holland (Altai Europe Ltd)